
Performance of a Rapid, 60 Second Multiplex Test for Simultaneous Detection of Antibodies to HIV-1, HIV-2 and T. pallidum in Serum, Plasma and Whole Blood

R. Galli, CDC Diagnostics Conference, Atlanta, March 2016



Review of INSTI® Immunofiltration Platform Attributes

INSTI HIV-1/HIV-2 Antibody Test



INSTI HIV Assay Characteristics

Performance Attribute	INSTI HIV-1/HIV-2 Antibody Test
HIV 1	Yes
HIV 2	Yes
Time to Results	1 minute
FDA approved	Yes
CLIA Waived	Yes
Assay description	IgM/IgG antibody sensitive flow-through rapid immunoassay
Early Detection	20-21 days post infection
Specimen Volume	50 µL

INSTI Multiplex HIV-1/HIV-2/Syphilis Antibody Test Development Objectives

Developed in response to growing need for integrated POC testing devices for sexually transmitted blood-borne infections (STBBI).




- Determine the performance of the *T. pallidum* assay in the INSTI Multiplex compared to reference methods for detection of *T. pallidum* antibodies.
- Preserve all design and performance attributes of the INSTI HIV-1/HIV-2 antibody assay
- Achieve performance attributes for *T. pallidum* similar to HIV

INSTI MULTIPLEX HIV-1/HIV-2/Syphilis Antibody Test

The **INSTI MULTIPLEX HIV-1/HIV-2/Syphilis Antibody Test** is a single use, rapid, flow-through *in vitro* qualitative immunoassay for the detection of antibodies to Human Immunodeficiency Virus Type 1, Type 2 and *Treponema Pallidum* in human EDTA-whole blood, fingerstick blood, serum or EDTA-plasma.

Multiplex is currently CE marked for sale and distribution in EU member countries; undergoing full dossier review in the WHO prequalification process.

INSTI Multiplex Test System

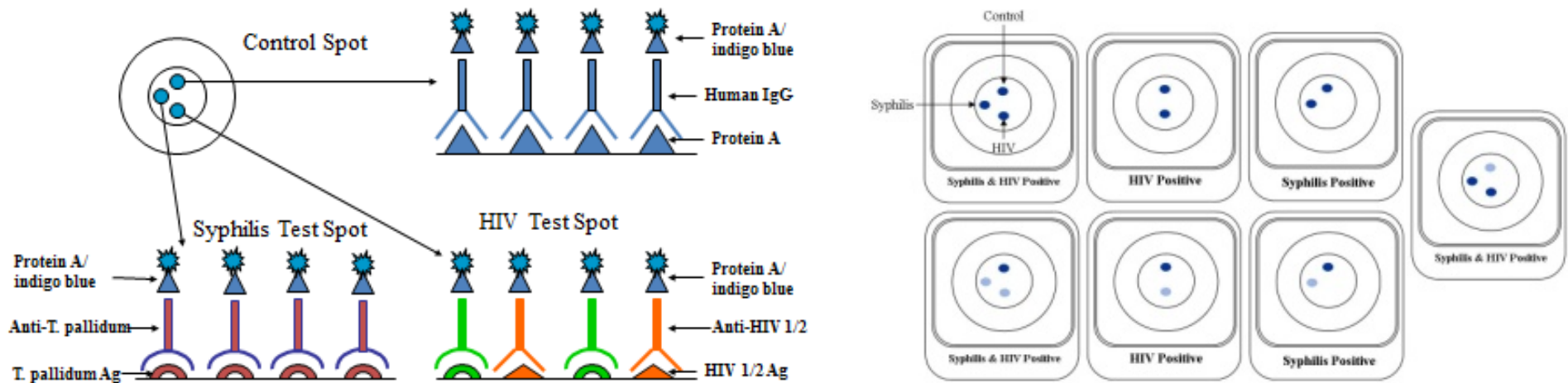

INSTI Multiplex Performance Data

Sensitivity of INSTI Multiplex for Detection of T. pallidum antibodies in a commercial syphilis mixed titre panel*.

	Panel Member ID Number									
COMPARATOR ASSAYS	1	2	3	4	5	6	7	8	9	10
Phoenix Biotech Trep Sure	P	P	P	N	N	N	N	N	P	P
Trinity Captia IgG	P	P	P	N	N	N	N	N	P	P
Trinity Captia IgM	N	N	N	N	N	N	N	N	P	N
Fisher Sureview RPR	P	P	P	N	P	P	P	N	P	P
Serodia-TP-PA	P	P	P	N	N	N	N	N	P	P
INSTI Multiplex HIV-1/HIV-2/Syphilis Antibody Test	P	P	P	N	N	N	N	N	P	P

* ZeptoMetrix Corporation

INSTI T. pallidum antibody detection was identical to other T. pallidum reference assays

INSTI Multiplex Performance Data

INSTI Multiplex T. pallidum antibody positive percent agreement with TP-PA assay, n=145 serum/plasma

INSTI Multiplex T. pallidum Antibody	TP-PA Final Interpretation		
	Positive	Negative	Totals
Reactive	138	0	138
Non-Reactive	7	0	7
Totals	145	0	145

The calculated positive percent agreement between INSTI Multiplex T. pallidum antibody detection and the approved TP-PA assay is **95.2%** ($138/145 = 95.2\%$, with 95% CI 90.4-97.6%).

INSTI Multiplex T. pallidum antibody negative percent agreement with TP-PA, n=379 serum/plasma

INSTI Multiplex T. pallidum Antibody	TP-PA Final Interpretation		
	Positive	Negative	Totals
Reactive	0	5	5
Non-Reactive	0	374	374
Totals	0	379	379

The calculated negative percent agreement between INSTI Multiplex T. pallidum antibody detection and the approved TP-PA assay is **98.7%** ($374/379 = 98.7\%$, with 95% CI 90.4-97.6%).

Performance of the INSTI Multiplex HIV 1/HIV2/Syphilis Antibody Test compared to TP-PA for Contrived Whole Blood Specimens (n=169).

INSTI Multiplex T. pallidum Antibody	TP-PA Final Interpretation		
	Positive	Negative	Percent Agreement
Reactive	62	0	Positive percent agreement 96.9% (62/64)
Non-Reactive	2	105	Negative percent agreement 100% (105/105)
Totals	64	105	169

Multiplex *T. pallidum* antibody sensitivity in clinically staged syphilis serum/plasma samples

Sensitivity of INSTI Multiplex for serum samples from patients with clinical syphilis diagnosis (n=96)*

Study Panel	INSTI <i>T. pallidum</i> Reactive	INSTI <i>T. pallidum</i> Non-Reactive	Sensitivity
Primary syphilis (n=33)	29	4	87.9% (29/33)
Secondary syphilis (n=41)	41	0	100% (41/41)
Latent syphilis (n=22)	21	1	95.5% (21/22)
Totals	91	5	94.8% (91/96)

* Dr. Anna Bianchi, Laboratoire départemental de Seine-Saint-Denis, Paris, France, 2011, personal communication

Performance of the INSTI Multiplex for detection of *Treponema pallidum* antibodies in patients with primary syphilis**

		T. pallidum reference test*		Total	Sensitivity
		Pos	Neg		
INSTI <i>T. pallidum</i>	Pos	11	0	11	78.6%
	Neg	3	0	3	
Total		14	0	14	

*Reference test was conducted using TP PCR DNA detection from lesions.

**Jeffrey D. Klausner, MD, MPH
Professor of Medicine and Public Health
UCLA David Geffen School of Medicine and Fielding School of Public Health, 2016, personal communication

INSTI Multiplex T. pallidum and HIV Results from a Field Study Conducted in Bangalore, India, 2012-2013, n=1010

T. Pallidum antibody performance of the INSTI Multiplex Test against the reference T. pallidum antibody status in a prospective field study population, n=1010*.

T. Pallidum antibody status	INSTI Multiplex T. pallidum Antibody Positive	INSTI Multiplex T. pallidum Antibody Negative
Positive	14	2
Negative	1	993
Total	15	995

*study population had a low prevalence of syphilis: 1.6% (16/1010)

HIV Performance of the INSTI Multiplex HIV-1/HIV-2/Syphilis Antibody Test against the reference HIV antibody status in prospective field study population, n=1010*

Panel	INSTI Multiplex HIV-1/HIV-2 Positive	INSTI Multiplex HIV-1/HIV-2 Negative
HIV antibody positive (n=136)	136	0
HIV negative (n=874)	874	0
Total	1010	0


HIV sensitivity: 100% (136/136)

HIV specificity: 100% (874/874)

*same study population had a high prevalence of HIV: 13.5%

Comparison of INSTI Syphilis kits before and after IgM removal for IgM + and IgG-/+ samples

- INSTI has been shown to have affinity to HIV gp41 IgM antibodies: *“Sensitivity of a rapid point of care assay for early HIV antibody detection is enhanced by its ability to detect HIV gp41 IgM antibodies.” N. Mosgabadi et al/Journal of Clinical Virology 71 (2015) 67-72*
- Recent data shows INSTI Multiplex also has affinity to T. pallidum p17/47 IgM antibodies:

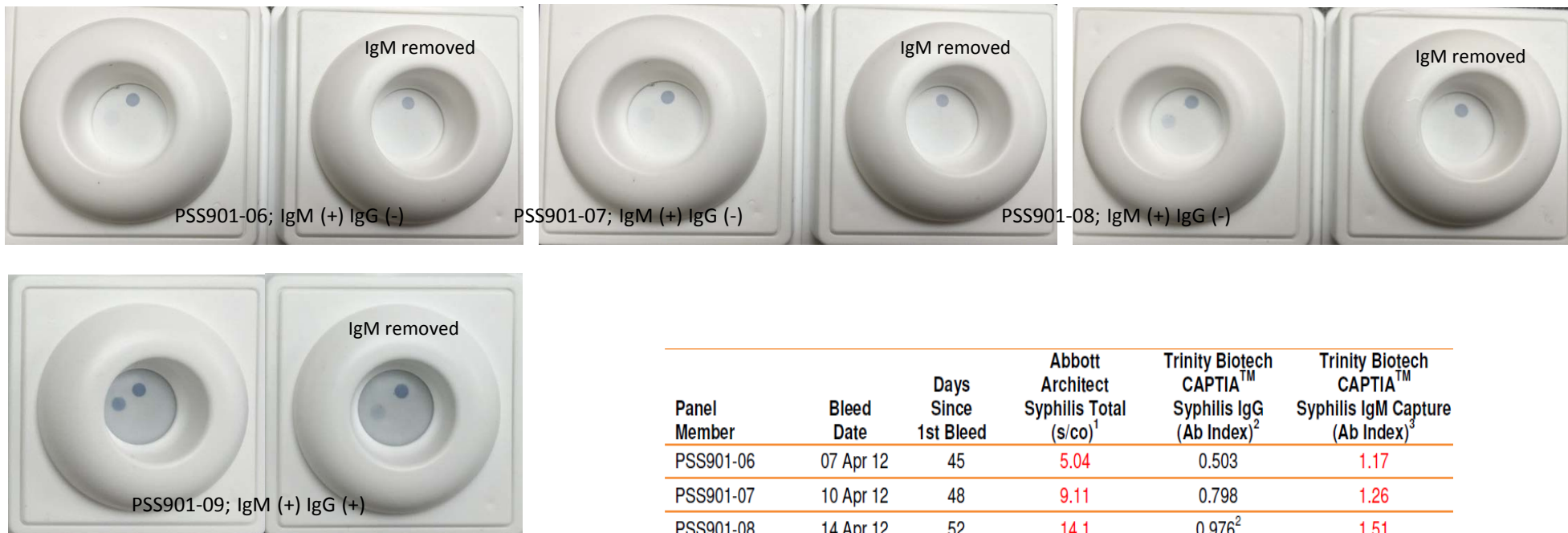

Panel Member	Blood Date	Days Since 1st Bleed	Abbott Architect Syphilis Total (s/co) ¹	Trinity Biotech CAPTIA™ Syphilis IgG (Ab Index) ²	Trinity Biotech CAPTIA™ Syphilis IgM Capture (Ab Index) ³
PSS901-06	07 Apr 12	45	5.04	0.503	1.17
PSS901-07	10 Apr 12	48	9.11	0.798	1.26
PSS901-08	14 Apr 12	52	14.1	0.976 ²	1.51
PSS901-09	21 Apr 12	59	27.4	1.32	1.87

Multiplex T. pallidum antibody assay replicates desirable performance attributes of INSTI HIV without affecting HIV performance

Performance Attribute	INSTI HIV-1/HIV-2 Antibody Test	INSTI Multiplex
HIV 1	Yes	Yes
HIV 2	Yes	Yes
T. pallidum	No	Yes
Time to Results	1 minute	1 minute
FDA Approved	Yes	No
CLIA Waived	Yes	No
Assay Description	HIV IgM/IgG antibody sensitive flow-through rapid immunoassay	HIV and T. Pallidum IgM/IgG antibody sensitive flow-through rapid immunoassay
Early Detection	20-21 days post HIV infection	85% sensitivity in primary syphilis
Specimen Volume	50 µL	50 µL

Summary and Conclusions

- **Performance of the INSTI Multiplex HIV-1/HIV-2/Syphilis Antibody Test for *T. pallidum* was similar to other *T. pallidum* antibody methods.**
- **As with other *T. pallidum* assays, lower sensitivity was observed in primary syphilis (85%) compared to secondary or latent syphilis (100%, 95.5% respectively).**
- **Field studies indicate that the performance for Multiplex HIV was not affected by the addition of the syphilis test spot (100% HIV sensitivity and specificity).**
- **INSTI Multiplex can detect IgM antibodies to *T. pallidum* p17/47 antigen epitopes.**